

Pahsco Vac Wound Drainage System

Step ①

Connect the drain to the entrance port of the reservoir.

Step ②

Whilst holding centre of reservoir with your left hand grip the bottom part of the reservoir with your right thumb and finger and pull gently towards yourself. Once you hear the click, the suction is activated.

Step ③

You should see drainage into the reservoir.

Step ④

When you need to empty the reservoir, loosen connector of outlet port, place fingers in centre of the reservoir (follow picture on reservoir). Gently squeeze down on the reservoir and pour contents into a safe container. z outlet port and follow again from step 2.



Pahsco Vac Wound Drainage System

Stock Code	Description
WD300315	Pahsco-Vac 150ml, Drain reservoir only
WD300330	Pahsco-Vac 300ml, Drain reservoir only
WD300345	Pahsco-Vac 450ml, Drain reservoir only