

# **SOL-JECT® Retractable Safety Syringe**

*We can eliminate syringe re-use and needlestick injury*

**A**



Draw the prescribed dosage into the barrel as you would with a conventional syringe with established protocols. **DON'T** push plunger to the end of barrel to avoid pre-activation.

**B**



Inject the medication into the patient. Push the plunger until there is an audible "click" as the plunger is engaged to the needle.

**C**



Withdraw needle from patient and then pull the plunger to its fully extended position. There should be a feeling of slight resistance as a locking ring at the end of the barrel secures the needle within the barrel.

**D**



Snap off syringes plunger at the breakpoint to permanently disable syringe. Discard immediately in an approved sharps container.

**Distributed By Akacia Medical:**

Tel: 021 5214100

Fax: 021 5214141

Email: [queries@akaciahealth.com](mailto:queries@akaciahealth.com)



## **Safety Message**

WHO recognized unsafe injection practices as an increasingly major source of infection with bloodborne pathogens such as HIV, HBV and HCV.

One child dies every 24 seconds from an unsafe injection. 64% of U.S. nurses reported being accidentally stuck by a needle while working.

Sol-Millennium Medical is dedicated to provide leading and innovative safe injection devices to people throughout the globe.

Sol-Ject® Retractable Safety Syringe is designed with advanced safety features to make it ideal for patients and healthcare professionals.